ENGLISH

CAUTION
FOR PROFESSIONAL DENTAL USE IN RECOMMENDED INDICATIONS ONLY (SEE 2. INDICATIONS FOR USE).

1. Product description

Ceramir® Crown & Bridge is a permanent, radiopaque, bioceramic luting agent. Ceramir® Crown & Bridge contains a bioceramic agent which hardens in the liquid, water, the cement, which hardens as a result of a chemical reaction, is self-setting.

In Ceramir® Crown & Bridge QuikCap, the cement is supplied in a capsule for mixing and direct application. The minimum volume of mixed cement from one capsule is 0.17 ml.

A capsule applicator and a mixing device are required for use. We recommend Ceramic Applicator and Ceramic Applicator 2 from Doxa Dental AB.

2. Indications for use

Ceramir® Crown & Bridge is intended for permanent cementation of:

- Metal crowns and bridges
- Gold crowns and onlays
- Cast or prefabricated metal posts
- High-strength ceramic crowns and bridges suitable for conventional cementation (e.g. zirconia, alumina, and lithium disilicate)

Contraindications

• Do not use as a direct pulp-capping agent. In case of close proximity to the pulp, do not exceed standard clinical procedures.

Precautions

Ceramir® Crown & Bridge contains polyacrylic acid, which may be irritating to eyes and skin. Avoid contact with eyes. In case of skin contact, rinse with plenty of water. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

• Do not use with patients with known allergy to polyacrylic acid. In rare cases, the product may cause sensitivity to the material. If such reactions are observed, discontinue the use of the product and consult a physician.

3. Step-by-step instructions

Note: Do not use any pre-treatment agents, e.g. silanes, primers or bonding agents in combination with Ceramir® Crown & Bridge.

Use of any such pre-treatments may compromise the bonding of the cement.

3.1 Tool preparation

Follow the manufacturer's instructions. Do not use any pre-treatment agents, e.g. silanes, primers or bonding agents.

3.2 Preparation of the restoration

Do not use any pre-treatment agents, e.g. silanes, primers or bonding agents when cementing with Ceramir® Crown & Bridge. Use of such agents will interfere with the bond between cement and restoration.

3.3 Pulp protection

• If pulp protection is required, use accepted standard clinical procedures.

3.4 Activation

ATTENTION: WORK QUICKLY
Start mixing immediately following activation (the reaction between the liquid and powder starts directly after activation).

• Open the aluminum pouch and remove the capsule.

• Hold the capsule firmly against a hard surface with the plunger facing down (fig. 1).

• Push down firmly, forcing the plunger into the body of the capsule until it stops (fig. 1).

3.5 Mixing

• Immediately insert the capsule into the applicator using the grooves to hold the plunger (fig. 2).

• Gently rotate the dosing lever (fig. 3a) two times to push the cement towards the plunger (fig. 3b).

• Then apply the cement by gently squeezing the lever two more times (fig. 3c).

3.6 Application

• Immediately insert the capsule in a high-frequency oscillating (4.000 to 5.000 rpm) or rotating capsule mixer (fig. 2).

• Mix for 8 seconds.

3.7 Luting and finishing

• Set the restoration and stabilize it under pressure, to avoid any movements, until the cement reaches a rubber-like consistency (approx. 2 min) (fig. 4).

• Remove the restoration and when the cement has reached the rubber-like consistency (fig. 5).

• Keep the restoration stabilized for an additional 1 minute (fig. 6).

• Check the margins of the restoration (especially the gingival) and remove any remaining cement.

• Advise the patient not to load their restoration for the first hour after cementation.

4. Cleaning the applicator

• Ceramic Applicator can be cleaned with standard cleaning agents, e.g. alcohol-based spray disinfectants

• Clean Ceramic Applicator 2 with alcohol-based spray disinfectant.

• Do not use any chemical sterilants or disinfectants.

AUTOCLAVING: Autoclave up to a maximum temperature of 140°C. Follow autoclave manufacturer's instructions for use. Do Not Ultrasonic Clean.

5. Storage and shelf life

• Keep Ceramic Crown & Bridge QuikCaps at temperatures between 4°C/39°F and 25°C/77°F. Do not open the aluminum pouch until immediately before use.

• If the product has been used strictly in accordance with these instructions for use, the shelf life is 2 years from the date of manufacture.

• **IMPORTANT:** Do NOT use the product after its expiry date.

• Check the expiration date on the product, including the 'VALIDITY' and 'EXPIRATION DATE'.

• Do not use Ceramic Crown & Bridge QuikCap capsules after their expiry date.

6. Limitation of Liability

If this product is found defective within the Warranty Period, your exclusive remedy and Doxa Dental AB's sole obligation and liability shall be the repair or replacement of the defective product. DOXA DENTAL AB WILL NOT BE LIABLE FOR ANY LOSS OR DAMAGE ARISING FROM THE PRODUCT, WHETHER DIRECT, INDIRECT, SPECIAL, PUNITIVE, INCIDENTAL, CONSEQUENTIAL OR OTHERWISE, OR FOR PROPERTY RIGHTS OR MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR NON-INFRINGEMENT OF A PATENT. EXCEPT AS PROVIDED IN THE WARRANTY, ALL IMPLIED WARRANTIES, INCLUDING WARRANTY, BREACH OF CONTRACT, NEGLIGENCE OR STRICT LIABILITY.

Trademarks

Ceramir® is a trademark of Doxa Dental AB.

Date of latest revision: 2016-01-14

Batch code **Radiopaque** **Capsule** **Applicator**

Do not use any pre-treatment agents, e.g. silanes, primers or bonding agents when in combination with Ceramic Crown & Bridge. Use of any such pre-treatments may compromise the bonding of the cement.

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